

Brown Stacking Commode with Removable Arms – 90323



Intended Purpose

Integrity Mobility Solutions commodes are designed to assist those who have difficulty in accessing or using a standard toilet. It can also be used as a temporary chair. All Integrity commodes have a hinged plastic toilet seat, commode bucket with lid and vinyl lift off seat.

Instructions for Use:

Remove vinyl seat to use commode.

Replace vinyl seat to use commode as a temporary chair.

- 1 - To adjust the height remove each 'E' clip in turn. Slide each leg to the required height.
- 2 - Align the holes and replace 'E' clip.
- 3 - Ensure that all legs are secured at the same position in order to prevent rocking.

Never extend legs in a manner so that the commode is slanting. This will compromise the safety of the user.

Detachable Arms Model

1. To detach arms for transfer, depress the metal buttons each side of the commode arms fixing sockets.
2. Using both hands, remove commode arm by lifting arm from fixing socket.
3. To refit arms, locate arms into fixing sockets and push down to stop.
4. Depress buttons on each end of arm, ensuring buttons "click" firmly into place.

Commode Buckets

To remove the commode bucket, simply lift the toilet's seat and fix the lid provided onto the bucket to avoid splashing and carefully raise the bucket and dispose of the contents in to a proper sluice or WC. Never use an open drain, sink or basin. This could cause a serious health risk. Clean out the residue with a suitable disinfectant and refit to the commode after use.

Cleaning

To clean vinyl seats use a suitable detergent and warm water, wipe seats and dry with disposable paper towel. Clean steel frame and toilet seat with a suitable detergent and warm water, wash framework and arm rests thoroughly paying particular attention to the toilet seat hinges and the underside of the toilet seat. Can be autoclaved at 80°C.

Allow equipment to dry prior to stacking and storage.

Please Note

Under no circumstances should the commode be moved whilst occupied. It is advised that you check the frame of the product annually for signs of wear and tear, dimpled or flaking paint or cracks in the frame should be reported to your supplier immediately. Should there be any such evidence, please refrain from using the commode.

Maintenance and Periodic Inspection

This product is maintenance free so long as the cleaning and safety instructions shown above are adhered to.

Check the product regularly for damage and to ensure that it is securely assembled.

Specification

Frame is composed of welded, Powder coated mild steel tube.

The seat pad is HF welded PVC with a foam and wood filling.

The extension legs are made from Chrome plated Mild Steel tube.

Load capacity: 180Kg

Weight of product: 9Kg

Dimensions

<u>Seat Pad Width</u>	16" (440mm)
<u>Seat Pad Depth</u>	16" (440mm)
<u>Seat Height (Including Seat Pad)</u>	19 ½" – 25 ½" (490mm – 640mm)
<u>Width between Armrests</u>	20" (505mm)
<u>Max. height</u>	36 ¾" (930mm)
<u>Min. Height</u>	30 ¾" (780mm)
<u>Max. Width</u>	21 ½" (550mm)
<u>Min. Width</u>	21 ½" (550mm)
<u>Max. Depth</u>	20" (505mm)
<u>Min. Depth</u>	18" (460mm)



Ives MD EU Rep. Ltd 44 Rosses Stables,
Ballymena, Co. Antrim, BT42 2SF

