












**KD GLIDEABOUT
COMMODE CHAIR
WITH GREY
WHEELS**

Instruction manual

REF.90519

Symbols appearing on the label or in the instruction manual

	Manufacturer address		Invitation to consult the user manual
	Date of production		Follow the safety instructions
	Batch number		CE conformity
	Product reference		Does not tolerate dampness
	Maximum user weight		

INSTRUCTION MANUAL

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESCRIPTION

The KD Toilet Wheelchair with grey wheels is a medical device. It is a technical toilet aid for the elderly, people with reduced mobility, disabled, injured or who can easily lose their balance.

ASSEMBLY PARTS



- | | | |
|----------------------|---------------------|------------------------|
| 1. Right Side Frame | 6. Backrest | 11. Left Footrest |
| 2. Left Side Frame | 7. Castors (Wheels) | 12. Right Footrest |
| 3. Connection Tubes | 8. Seat Assembly | 13. Spanner |
| 4. Right Armrest | 9. Potty & Lid | 14. Backrest Pad Allen |
| 5. Left Armrest | 10. Seat Pad | Key |
| 15. Castor Allen Key | 16. E-Clips | |

SAFETY PRECAUTIONS



Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 165 kg.
- Check all components are secured before use.
- Make sure that the product is in good working order.
- Visually check the product.
- Before each use of the product, the brakes and castors should be checked.
- Check the stability of the chair before use.
- Only use on a level ground.
- When stationary, apply the brakes on the castors.
- Ensure the brakes are released before the chair is moved again.
- Always ensure that the device is stationary and brakes applied when transferring to and from the device. Swing away the footrests before getting in or out of the device. The user should always remove their feet from the footrests before swinging the leg rests away.
- Do not lean out of the device.
- Be careful of pinch points when rising from or lowering on to the seat, or when you are removing the potty.
- Make sure the potty rests securely on the two rails under the seat.
- Check, the castors for wear, the castors bearings, the brakes, the footrest locking mechanism, the seat and backrest, the arm Pads, the potty and the lid.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING & USE

ASSEMBLING THE CHAIR FRAME

- Take one of the Side Frames (Items & 2) along with the connection Tubes (Item 3).
- Push the longest tube onto the shorter of the two tubes sticking up from the Side frame, aligning the hole with the spring button.
- Press the spring button down to allow the Connection Tube to fully lock in place.
- Repeat for the short Connection Tube on the longer Side Frame tube.
- Repeat the process with the second Side Frame.
- When complete, the frame should appear as shown below.




ARMRESTS OPERATION



- The armrests (Items 4 & 5) can be removed for a lateral transfer and replaced if necessary by the user.
- To replace the armrests, push the two ends of the armrest tubing into the receivers located on the frame.
- Press the spring button each end to lock the armrest into the frame.
- To remove the armrests, simply reverse the previous instruction

BACKREST INSTALLATION

The Backrest will easily slide into the tubes that are found at the rear of the completed frame assembly. Once the holes are aligned, push the e-clips Item 16 through the holes until the clip locks onto the outside of the tube & the pin is showing on the opposite side of the tube.

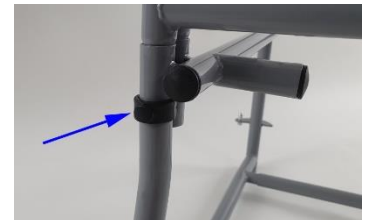

- Ensure that the Backrest handle is facing away from the frame when fitted.

CASTOR INSTALLATION


Each castor (Item 7) is supplied with a screw & nut in the castor stub.

- Before attempting to fit the castor to the frame, remove the nut & screw.
- Once the nut & screw are removed, place the stub of the castor into the upturned frame assembly, making sure the holes in the stub & the frame are aligned.
- Push the screw through the frame & stub, then tighten the nut by hand. Once finger tight, securely tighten the nut with the Allen Key (Item 15) & Spanner (Item 13) supplied.
- Repeat the above steps for
- The remaining three castors.




FOOTRESTS INSTALLATION

- To put the footrests in place, insert the 2 tubing holes in the lugs when the footrest is rotated outwards. Then rotate the footrest as shown in the diagram to lock it in the use position at the front of the chair.
- To rotate the footrest and retract it, press the chrome lever while pushing back the footrest. The footrest can then be lifted to be removed.

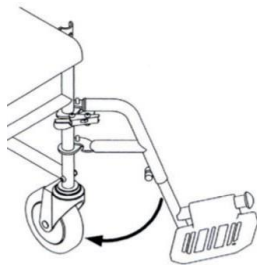

USE OF BRAKES ON WHEELS

Press the lever located above the wheel to apply the parking brake (as indicated on the diagram). To release the lock, press the protruding tab above the wheel.

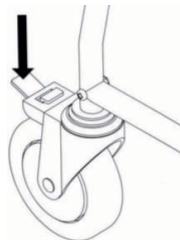

SEAT INSTALLATION

This product has a blow moulded seat which is easily clipped onto the frame. The seat also has guide rails fitted for the fitting of the potty & lid beneath the seat.

- Line up the seat over the frame, so that the clips match the positions of the frame tubes.
- Push the seat down onto the frame, ensuring that all of the clips are secure and tight on the tubes of the frame.


Once all of the clips are secure, the potty can be slid onto the guides beneath the seat.



USE OF POTTY AND LID

- The potty and lid are housed under the plastic seat. To add the potty, slide under the seat from the back of the chair. Make sure the potty is well fixed on the two rails fixed below from the seat.
- To remove the potty, slide the potty under the seat. Add the lid and carry the potty safely using the chrome handle.

CARE AND MAINTENANCE

 Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product (all castors, seals, and fixings, pot and lid) with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The Polyurethane (PU) and plastic parts of the commode can be cleaned in an aqueous decontaminant machine at 80°C. Cleaning the aluminium frame in this way is not recommended as it could cause damage to paintwork.
- The chair must be completely overhauled every year, with compulsory lubrication.
- It is recommended that the device is serviced at least annually.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
- Worn tyres, wheel bearings, castors, brakes, leg rest locking mechanism, seat and back cushions, armrest cushions, quick release pin, pot and lid, «E» clips) should be repaired, replaced, adjusted and/or lubricated if necessary.

- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.


CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

 Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
 - Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
 - All components must be stored without any load (do not place any objects that are too heavy on the components of the product, do not jam anything.)

PERIOD OF USE

The life expectancy of this product is 6 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL FEATURES

Overall dimensions	width 54 cm x depth 93 cm x height 98 cm
Seat Dimensions	width 44 cm x depth 46 cm x height 50 cm
Backrest height	42 cm
Armrest height	74 cm
Potty capacity	5 litres
Weight	14 kg
Maximum weight authorized	165 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. Integrity Mobility Solutions will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.



Integrity Mobility Solutions
74 St. Georges Road
Bolton BL1 2DD



Maximum weight authorized
165 kg



Ives MD EU Rep. Ltd 44 Rosses Stables,
Ballymena, Co. Antrim, BT42 2SF

